

DryEars

DryEars Spray is an ear-water removal and drying aid product formulated to relieve ear discomfort due to trapped ear-water in the external ear canal after aquatic activity.

AquaFree Ears Spray, prevents harmful effects such as bacteria or fungal infections and pain.

DryEars Spray combines a unique formula in a spray device.

When should DryEars Spray be used?

After water activities like, swimming, diving, bathing or showering.

How does DryEars Spray work?

The **DryEars Spray** formula has special physical-chemical characteristics that are activated when in contact with water. The unique formulation mixed with the trapped water in the ear, reduces its specific gravity enabling fast and full evaporation of the water and moisture from the external ear canal.

The micro spray mechanism allows optimal dispersion of **DryEars Spray** in the ear for immediate result.

Ingredients: Isopropyl alcohol, Glycerin, Rose oil.

Directions for Use: Apply a single spray in affected ear. If needed, repeat use.

Warnings:

- **DryEars Spray** should not be used in patients with perforated eardrum
- Keep out of reach of children
- Avoid contact with the eye
- For external use only

Storage Conditions: Store in a cool place, do not expose to temperature over 50°C. Do not throw into fire.

Managing empty spray containers: DryEars containers are exempt from hazardous waste rules. They have no special storage, labeling or disposal requirements. Recycle them, if possible or send them to an incinerator that will recover the metal.



Naveh-Pharma LTD.

11, Yad-Harutzim St., Netanya, 4250519 Israel
www.navehpharma.com



OBELIS S.A. 53 Boulevard
Général Wahis 1030 Brussels, Belgium

The lifetime of the device 36 months from the manufacturing as stated on the label