

SalinaGel

A Gentle Moisturizing Gel for Dry Nasal Passages

SalinaGel is an isotonic water based saline nasal moisturizing gel that relieves dry, irritated nasal passages caused by cold and allergy medications, dust and low humidity.

About Dryness of the Nostrils

The nasal wall is composed of epithelial cells, named mucosa. The main functions of these cells are to secrete mucus to humidify and filter inhaled air and assist in preventing the entry of infectious agents such as dust, pollen pollution and bacteria. These agents stick to the mucosa, preventing a smooth inhalation process throughout the nasal passageway.

Under certain conditions the ability of the nasal epithelial cells to secrete mucus and filter inhaled air may be impaired resulting in a sense of dryness in the nose causing discomfort, pain or sneezing. This is due to the penetration of dust to the lower respiratory airways. Moreover, infectious agents may have entered into the nasal airways increasing the possibility of infections.

SalinaGel Action

In order to preserve nasal moisture and prevent dry and irritated nasal passageways caused by the daily exposure to dust, air pollution, air-conditions, etc., **SalinaGel** does the following triple action:

1. moisturizes, nasal airways
2. relieves dryness and irritation
3. restores the filter and humidifying functions of the mucosa cells.



SalinaGel ensures the normal function and wellness of the nasal cavities.

It contains sea salt in isotonic concentration (similar to the salt concentration in the healthy tissue) and moisturizing agents that preserve the healthy function of mucosa and restore the system's ability to filter and treat inhaled air through the nose.

Composition:

Purified water, Hydroxyethylcellulose, Sodium chloride, Propylen Glycol, D-Sodium EDTA, Panthenol, Alantoin, Menthol Crystals, Sodium Hyaluronate, Glycerine, Perfume.

Indications:

SalinaGel moisturizes dry nasal passages and relieves dry, irritated nasal passages.

Directions for Use:

Gently blow your nose. Apply a small amount of **SalinaGel** and rub gently inside the nostril. You can massage the nostril externally in order to disperse the gel.

Suitable for babies from 6 months

Dosage:

Apply as needed.

Warnings:

Keep out of reach of children.
Avoid contact with eyes.
For external use only.

Storage:

Store in a cool and dry place, do not refrigerate.



Naveh-Pharma LTD.

11, Yad-Harutzim St., Netanya, 4250519 Israel
www.navehpharma.com



OBELIS S.A. 53 Boulevard
Général Wahis 1030 Brussels, Belgium

The lifetime of the device 36 months from the manufacturing as stated on the label

Version 3 5/2021