

A circular icon containing a black virus particle with white spikes, crossed out by a grey diagonal line.

VirusStop®



Nasal Gel - Prevents the penetration of viruses through the nose into the respiratory system

What is VirusStop®? VirusStop® is an aqueous gel consisting of High molecular weight hyaluronic acid and cellulose. When VirusStop® applied in the nostrils, the gel spread on the walls of the nose, capturing most of the viruses, disrupting its outer layer.

VirusStop® block viral growth and consequently preventing flu.

Antiviral substances, such as bleach, destroy viruses with simple contact, but they cannot be applied to the human body without causing severe damage. Most of the antiviral drugs today work by inhibiting the development of viruses but are unable to destroy them, moreover viruses can change and become resistant to such treatments.

In order to overcome these two obstacles and be able to fight viral infections effectively, Naveh-Pharma has found a unique mechanism to stop droplets transmission by creating protective film of **VirusStop®** gel that acts as barrier to which viruses attached and neutralized.

VirusStop® is non-toxic and does not contain preservatives.

VirusStop® gel for the whole family.

VirusStop® may be effective against Covid-19.

Instruction for Use: Apply 1 drop of **VirusStop®** gel in each nostril.

VirusStop® is effective for several hours in preventing droplet transmission, therefore, it is recommended to apply **VirusStop®** always before entering an environment at high risk of infection, such as public-transport, airplane, sporting events, theaters, crowded places and before entering enclosed spaces or social gatherings people.

Ingredients: High molecular weight sodium hyaluronate, Hydroxyethylcellulose, Propylen Glycol, Sodium EDTA, DL-Panthenol, Allantoin, Sodium Chloride, Glycerin, Purified water.


Warnings: Keep away from children. Avoid contact with eyes (if accidental eye contact occurs, rinse with water).

For external use only.

Storage: Store in a cool, dry place.

Do not refrigerate.

Suitable for babies from 6 months.

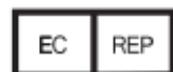

OBELIS S.A.

53 Boulevard Général Wahis
1030 Brussels, Belgium

The lifetime of the device 36 months from the manufacturing as stated on the label

Version 3 5/2021



Manufacturer: Naveh-Pharma Ltd.,

P.O. Box 8139, Netanya 4250519, Israel.

Tel. 972-98810000 www.navehpharma.com

