

Medical Device

15mL

For Ear Wax Removal Wash & Dissolve Jet Spray

- For ear wax removal
- Double action Wash & Dissolve system
- Prevents ear plugging
- Safe & effective
- User friendly
- Clinically tested
- No preservatives

Directions:



Spray 1-2 puffs in each ear 3-5 times a day until the wax dissolves.

Ingredients: Mineral Oil, Phytosqualan, Spearmint Oil.

FOR EXTERNAL USE ONLY



Naveh-Pharma LTD.

11, Yadi-Harutzim St., Netanya, 4250519 Israel

www.navehpharma.com



OBELIS S.A. 53 Boulevard
Général Wahis 1030 Brussels, Belgium



LOT



Warnings:

Should not be used in cases of perforated eardrum and grommet

Do not spray into the eye

Keep out of reach of children



Clean Ears

NAVEH
PHARMA